

Designed

with your needs in mind



CrystalClear Plus

Patients and support networks in Home Care

Rüsch CrystalClear Plus Tracheostomy Tubes have been designed to provide comfort. The thermo-sensitive PVC tube has sufficient rigidity for insertion, then softens at body temperature to accommodate anatomical differences.

A Rüsch CrystalClear Plus flexible inner cannula can be removed or replaced freely to reduce the risk of accidental decannulation and performance of the tube is not compromised when in situ, again with comfort in mind.

No cleaning required as the kit contents are discarded after use. Replacement inner cannula, neck band and accessories are available separately for your convenience.


How were your needs considered in the CrystalClear Plus product design process?

- CrystalClear Plus accessories are only used when breathing is not assisted- not connected to breathing apparatus.
- Accessories include a cough cap to direct coughed sputum downward into a tissue, a sealing cap that occludes the tube connector for oral/nasal breathing and a shower cap to reduce the risk of water droplets entering the tube.
- Flexible neck flange and a soft neck band reduces movement of the tube during use for additional comfort.
- The thin walled cuffed and un-cuffed tracheostomy tubes are available in sizes 3.5mm to 10.0mm (including all half sizes) to ensure the best fit and support breathing effort.
- Size 6 and above kits include 2 flexible inner cannulae.

Rüsch CrystalClear Plus Tracheostomy Tube

15 mm Swivel Connector
Designed to reduce torque with ventilator circuits

Cuff
Tubes are available both with or without a cuff

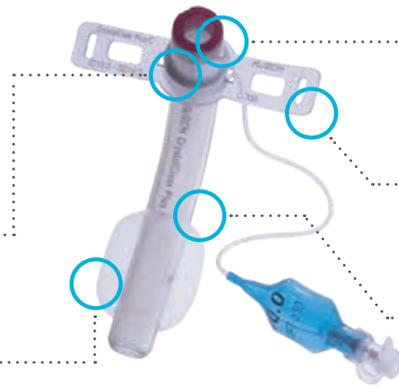

Inner Cannula
Hub colour of inner cannula matches font colour on fixation flange

Thin, Flexible Flange
Facilitates effective stoma care¹

95° Shaft
Promotes a secure fit to help prevent accidental decannulation²

Rüsch CrystalClear Plus Tracheostomy Tube

PRODUCT CODE* UNITS OF 5	PRODUCT CODE* UNITS OF 1	PRODUCT CODE** UNITS OF 5	PRODUCT CODE** UNITS OF 1	SIZE / I.D. (MM)	O.D. (MM)	LENGTH (MM)	ANGLE Ø APPROX.	CUFF Ø (MM)	INNER CANNULA	TUBE	COLOUR CODE	QTY
121300-000035	858600-000035	121303-000035	858603-000035	3.5	5.3	47.0	95°	11.0	-	3.5	Red	5
121300-000040	858600-000040	121303-000040	858603-000040	4.0	5.9	54.0	95°	12.0	-	4.0	White	5
121300-000045	858600-000045	121303-000045	858603-000045	4.5	6.5	54.0	95°	14.0	-	4.5	Grey	5
121300-000050	858600-000050	121303-000050	858603-000050	5.0	7.0	57.0	95°	14.0	-	5.0	Blue	5
121300-000055	858600-000055	121303-000055	858603-000055	5.5	7.5	58.0	95°	15.0	-	5.5	Green	5
121300-000060	858600-000060	121303-000060	858603-000060	6.0	8.7	64.0	95°	19.0	6.0	-	Orange	5
121300-000065	858600-000065	121303-000065	858603-000065	6.5	8.7	64.0	95°	19.0	6.5	-	Maroon	5
121300-000070	858600-000070	121303-000070	858603-000070	7.0	10.0	70.0	95°	21.0	7.0	-	Blue	5
121300-000075	858600-000075	121303-000075	858603-000075	7.5	10.7	75.5	95°	25.0	7.5	-	Yellow	5
121300-000080	858600-000080	121303-000080	858603-000080	8.0	11.3	81.0	95°	28.0	8.0	-	Lilac	5
121300-000085	858600-000085	121303-000085	858603-000085	8.5	11.3	81.0	95°	28.0	8.5	-	Green	5
121300-000090	858600-000090	121303-000090	858603-000090	9.0	12.7	87.0	95°	28.0	9.0	-	Brown	5
121300-000095	858600-000095	121303-000095	858603-000095	9.5	12.7	87.0	95°	28.0	9.0	-	Navy	5
121300-000100	858600-000100	121303-000100	858603-000100	10.0	14.0	95.5	95°	30.0	10.0	-	Red	5

*Without Cuff. **With Cuff.

Kit contains: Tracheostomy Tube (cuffed or un-cuffed, Soft Neck Band, 2 colour coded inner cannulae, Patient card, Sachet of water based lubricant gel

Rüsch CrystalClear Accessories

PRODUCT CODE		DESCRIPTION	QTY
121311		Shower cap for Rüsch CrystalClear Plus	4
121312		Cough cap for Rüsch CrystalClear Plus	4
121313		Sealing cap for Rüsch CrystalClear Plus	4
507900		Comfort Neck Band	10
507800		Comfort Neck Band Pediatric	10

Rüsch CrystalClear Inner Cannula

PRODUCT CODE	SIZE	I.D. (MM)	O.D. (MM)	COLOUR CODE	QTY
121310-000060	6.0	5.0	5.7	Orange	10
121310-000065	6.5	5.0	5.7	Maroon	10
121310-000070	7.0	6.0	6.7	Blue	10
121310-000075	7.5	6.5	7.3	Yellow	10
121310-000080	8.0	7.0	7.7	Lilac	10
121310-000085	8.5	7.0	7.7	Green	10
121310-000090	9.0	8.0	8.7	Brown	10
121310-000095	9.5	8.0	8.7	Navy	10
121310-000100	10.0	8.8	9.7	Red	10

www.teleflextracheostomy.com



References

1. Wilkinson KA, Martin IC, Freeth H, Kelly K, Mason M. NCEPOD: On the right Trach? 2014. Available at: <http://bit.ly/ontherighttrach>. Accessed August 22, 2017.
2. Mallett J, Albarran JW, Richardson A, eds. Critical Care Manual of Clinical Procedures and Competencies. Oxford, UK: Wiley Blackwell; 2013.

Please ask your Teleflex Sales Representative if you wish to know more or visit our Rüsch Tracheostomy micro site where you will find out more about this range and our other Rüsch exciting products. Contents of unopened, undamaged package are clean, ready to use and for single-use. Refer to package insert for current warnings, indications, contraindications, precautions, and Instructions for Use.

Teleflex, the Teleflex logo, CrystalClear and Rüsch are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. Information in this document is not a substitute for the product Instructions for Use. The products in this catalogue may not be available in all countries. Please contact your local representative. Subject to technical changes without further notice. Revised 02 2023

© 2023 Teleflex Incorporated. All rights reserved. MCI-2023-0075 · REV 0 · 02 23 PDF

Teleflex

Distributed by:
Teleflex Headquarters International, Ireland · Teleflex Medical Europe Ltd. · IDA Business and Technology Park
Dublin Road · Athlone · Co Westmeath · Tel. +353 (0)9 06 46 08 00 · Fax +353 (0)14 37 07 73 · orders.intl@teleflex.com
United Kingdom Tel. +44 (0)14 94 53 27 61 · info.uk@teleflex.com
South Africa Tel. +27 (0)11 807 4887 · assist.africa@teleflex.com