

Designed

with your needs in mind



TracFlex Plus

Patients and support networks in Home Care


Teleflex have continued to improve our offering over the past 35 years, to now offer our newly upgraded range of TracFlex Plus adult cuffed and un-cuffed tracheostomy tube sets.

When designing tracheostomy tubes to meet the needs of patients in alternate care, these are core “must have” benefits:

- Comfort: tube, cuff (if required), neck flange and neck band
- Should allow optimal breathing with a minimal OD/ID tube ratio
- Provide dedicated accessories to enhance safety and daily living
- Tube can be cleaned if required to enhance hygiene routine
- Inner cannula to keep the main tube clean should be flexible to avoid affecting the tube’s performance.

How were your needs considered in the TracFlex Plus product design process?

- Highly flexible/non kink tubing, soft neck band and pliable fixation wings assist in comfort during ventilation/spontaneous breathing eating, drinking and natural head movements.
- The adjustable flange moves freely when unlocked allowing for safe readjustment during and after stoma maintenance.
- An audible click confirms that the adjustable flange mechanism is securely locked to reduce the risk of accidental de-cannulation.
- Neck fixation wings are clear to show natural skin tone.
- Flexible inner cannula, Shower, Cough and Sealing caps available

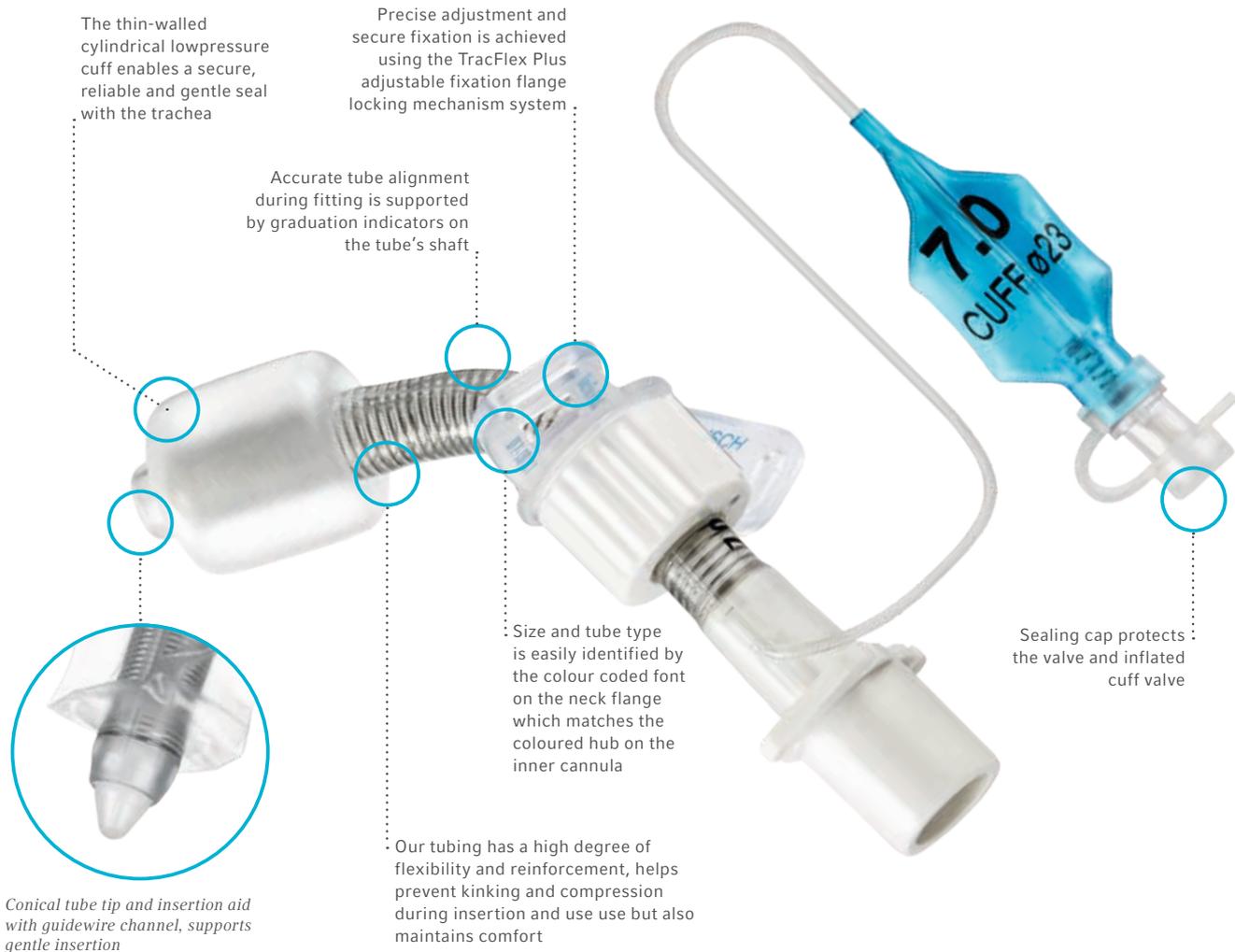

The thin-walled cylindrical lowpressure cuff enables a secure, reliable and gentle seal with the trachea

Precise adjustment and secure fixation is achieved using the TracFlex Plus adjustable fixation flange locking mechanism system

Accurate tube alignment during fitting is supported by graduation indicators on the tube's shaft

Size and tube type is easily identified by the colour coded font on the neck flange which matches the coloured hub on the inner cannula

Sealing cap protects the valve and inflated cuff valve

Conical tube tip and insertion aid with guidewire channel, supports gentle insertion

Our tubing has a high degree of flexibility and reinforcement, helps prevent kinking and compression during insertion and use but also maintains comfort

TracFlex Plus Set (with Cuff, Ref. 121903; without Cuff, Ref. 121900)

ORDER SIZE / I. D.	O.D. IN MM	LENGTH (CL) RANGE IN MM	DIM. A IN MM	DIM. B IN MM	DIM. C IN MM	ANGLE θ (APPROX.)	CUFF Ø IN MM **
● 7.0	10.1	60.3 - 66.5	37.5	6.2	22.8	100°	23
● 8.0	11.1	66.9 - 88.5	41.5	21.6	25.4	100°	27
● 9.0	12.1	70.4 - 117.3	44.5	46.4	26.4	100°	30
● 10.0	13.1	78.5 - 121.3	44.5	42.8	34.0	100°	32
● 11.0	14.2	80.6 - 125.3	47.5	44.7	33.1	100°	34

** Only cuffed versions

TracFlex Plus Inner Cannulas (Ref. 121910)

ORDER SIZE	I.D. IN MM	O.D. IN MM	QTY
● 7.0	5.5	6.2	10
● 8.0	6.5	7.2	10
● 9.0	7.8	8.3	10
● 10.0	8.8	9.4	10
● 11.0	9.8	10.4	10

Our Flexible Inner Cannula System: Effective Secretion Management Made Easy

TracFlex Plus inner cannula is designed to protect the inner lumen of the tracheostomy tube against the build up of secretions. Single use to support good hygiene, reducing the risk of infection to patients and carers.

The TracFlex Plus thin walled, flexible, inner cannula enables a close fit to the inner diameter of the tracheostomy tube, with minimal torque on the tube during insertion and removal of the inner cannula.

TracFlex Plus Accessories

REF.	DESCRIPTION	QTY
121911	Shower cap for TracFlex Plus tubes, acting as a splash guard to prevent the entry or inhalation of water droplets during bathing or showering	4
121912	Cough cap for TracFlex Plus tubes, to channel expelled secretions downwards	4
121913	Sealing cap for TracFlex Plus tubes, for weaning therapy	4

Please ask your Teleflex Sales Representative if you wish to know more or visit our Rüsch Tracheostomy microsite where you will find out more about this range and our other Rüsch exciting products.

Teleflex, the Teleflex logo, Rüsch, and TracFlex Plus are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. Information in this document is not a substitute for the product Instructions for Use. The products in this catalogue may not be available in all countries. Please contact your local representative. Revised 12 22. Subject to technical changes without further notice. © 2023 Teleflex Incorporated. All rights reserved. MCI-101149 · REV 0 · 02 23 PDF