

# Designed

*with your needs in mind*



## TracheoFix

### Clinicians Acute Care, ENT, OR

Rüsch TracheoFix single patient use tracheostomy tubes have been designed with long term patient use in mind. Manufactured in thermosensitive PVC, the semi rigid design assists in tube insertion<sup>1</sup> but will soften at body temperature to help accommodate a patient's anatomy. The thin walled, curved tube design, soft secure neck band and a flexible fixation wing system helps to provide patient comfort as well as supporting the effort of breathing. The discreet connection system securely attaches to a range of dedicated accessories for spontaneously breathing patients use.

Speaking valve has an optional O<sub>2</sub> adapter to provide a bolus of oxygen during inspiration. There is also choice of inner cannula designs, either with or without a 15mm connector pre-attached for use with all TracheoFix Tracheostomy tubes.

#### How were the needs of your profession considered in the TracheoFix product design process?

- Flexible neck flange and a soft neck band helps reduce movement of the tube against granulating tissue during use, which can assist in minimising the risks associated with bleeding, infection and patient discomfort.<sup>2</sup>
- Each inner cannula has a coloured ring to indicate which tube size it should be used with, and the corresponding tube uses the same colour for the information print on the fixation flange. This reduces the risks of accidentally using a smaller inner cannula increasing the effort of breathing.
- Kits contents are dependant on the tube selected for use and all accessories/inner canula are available to order separately.

# TracheoFix Tracheostomy Tube

## Inner Cannula



### Colour coding

Size-specific colour coded font to aid in size identification for inner cannula

### Fenestrated kits

Fenestrated and non fenestrated tube options

### Cuff

Tubes are available both with or without a low pressure cuff



### Thin, Flexible Flange

Facilitates effective stoma care and gives the option of using both sutures and a neck band to secure

### Accessories



## Rüsch TracheoFix Tracheostomy Tube

ORDER NUMBER 5 UNITS	REF	SIZE / I.D. (MM)	O.D. (MM)	LENGTH (MM)	ANGLE Ø APPROX.	CUFF Ø (MM)	KIT CONTENTS
121400-000070	TFC	7.2	9.0	70.0	102°	27.0	Cuffed Tube, comfort neck band, 2x 15mm inner cannula, insertion aid (Obturator).
121400-000085	TFC	8.7	10.3	78.0	94°	27.0	
121400-000100	TFC	10.2	12.3	86.0	90°	32.0	
121400-000110	TFC	11.2	13.3	86.0	92°	35.0	
121410-000070	TFCP	7.2	9.0	70.0	102°	27.0	Cuffed Tube, insertion aid, comfort neck band, 1x 15mm inner cannula with fenestrations, 1 inner cannula without fenestrations, 1x low profile inner cannula with fenestrations and 1 inner cannula without fenestrations, fixation ring, cough cap, speaking valve with O <sub>2</sub> adapter.
121410-000085	TFCP	8.7	10.3	78.0	94°	27.0	
121410-000100	TFCP	10.2	12.3	86.0	90°	32.0	
121410-000110	TFCP	11.2	13.3	86.0	92°	35.0	
121415-000070	TF	7.2	9.0	70.0	102°	-	Uncuffed Tube, insertion aid, comfort neck band, 1x 15mm inner cannula, 1x low profile inner cannula, fixation ring, cough cap.
121415-000085	TF	8.7	10.3	78.0	94°	-	
121415-000100	TF	10.2	12.3	86.0	90°	-	
121415-000110	TF	11.2	13.3	86.0	92°	-	
121420-000070	TFP	7.2	9.0	70.0	102°	-	Uncuffed Tube, insertion aid, comfort neck band, 1x 15mm inner cannula with fenestrations, 1 inner cannula without fenestrations, 1x low profile inner cannula with fenestrations, 1 inner cannula without fenestrations, fixation ring, cough cap, speaking valve with O <sub>2</sub> adapter.
121420-000085	TFP	8.7	10.3	78.0	94°	-	
121420-000100	TFP	10.2	12.3	86.0	90°	-	
121420-000110	TFP	11.2	13.3	86.0	92°	-	

## Rüsch TracheoFix Inner Cannula

ORDER NUMBER 5 UNITS	I.D. (MM)	O.D. (MM)	DESCRIPTION
121425-000070	5.5	6.7	15mm connector pre-attached
121425-000085	7.0	8.2	15mm connector pre-attached
121425-000100	8.5	9.7	15mm connector pre-attached
121425-000110	9.5	10.7	15mm connector pre-attached
121430-000070	5.5	6.7	Low profile (short)
121430-000085	7.0	8.2	Low profile (short)
121430-000100	8.5	9.7	Low profile (short)
121430-000110	9.5	10.7	Low profile (short)
121435-000070	5.5	6.7	Fenestrated with 15mm connector pre-attached
121435-000085	7.0	8.2	Fenestrated with 15mm connector pre-attached
121435-000100	8.5	9.7	Fenestrated with 15mm connector pre-attached
121435-000110	9.5	10.7	Fenestrated with 15mm connector pre-attached
121440-000070	5.5	6.7	Low profile with fenestrations
121440-000085	7.0	8.2	Low profile with fenestrations
121440-000100	8.5	9.7	Low profile with fenestrations
121440-000110	9.5	10.7	Low profile with fenestrations

## Rüsch TracheoFix Accessories

PRODUCT CODE	DESCRIPTION	QTY
121445	Sealing cap	4
121455	Speaking valve with O <sub>2</sub> adapter	2
121460	Cough Cap	4
121465	Fixation ring for use with low profile inner cannula	4



[www.teleflextracheostomy.com](http://www.teleflextracheostomy.com)

### References

1. Wilkinson KA, Martin IC, Freeth H, Kelly K, Mason M. NCEPOD: On the right Trach? 2014. Available at: <http://bit.ly/ontherighttrach>. Accessed August 22, 2017.
2. Mallett J, Albarran JW, Richardson A, eds. Critical Care Manual of Clinical Procedures and Competencies. Oxford, UK: Wiley Blackwell; 2013.

Please ask your Teleflex Sales Representative if you wish to know more or visit our Rüsch Tracheostomy micro site where you will find out more about this range and our other Rüsch exciting products. Contents of unopened, undamaged package are clean, ready to use and for single-use. Refer to package insert for current warnings, indications, contraindications, precautions, and Instructions for Use.

Teleflex, the Teleflex logo, Rüsch, and TracheoFix are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. Information in this document is not a substitute for the product Instructions for Use. The products in this catalogue may not be available in all countries. Please contact your local representative. All data current at time of printing. Revised (01/2023). Subject to technical changes without further notice.

© 2023 Teleflex Incorporated. All rights reserved. MCI-2023-0071 · REV 0 · 01 23 PDF

**Teleflex**

### Distributed by:

Teleflex Headquarters International, Ireland · Teleflex Medical Europe Ltd. · IDA Business and Technology Park  
Dublin Road · Athlone · Co Westmeath · Tel. +353 (0)9 06 46 08 00 · Fax +353 (0)14 37 07 73 · [orders.intl@teleflex.com](mailto:orders.intl@teleflex.com)  
United Kingdom Tel. +44 (0)14 94 53 27 61 · [info.uk@teleflex.com](mailto:info.uk@teleflex.com)  
South Africa Tel. +27 (0)11 807 4887 · [assist.africa@teleflex.com](mailto:assist.africa@teleflex.com)